User Manual

DT-215P-M1

Medical-Grade Desktop Monitor

About This Document

No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system, or translated into any language or computer language, in any form or by any means, including, but not limited to, electronic, magnetic, optical, chemical, manual, or otherwise without prior written permission of MicroTouch™ a TES Company.

The information in this document is subject to change without notice. MicroTouch™ a TES Company makes no representations or warranties with respect to the contents herein, and specifically disclaims any implied warranties of merchantability or fitness for a particular purpose. MicroTouch™ a TES Company reserves the right to revise this publication and to make changes from time to time in the content hereof without obligation of MicroTouch™ a TES Company to notify any person of such revisions or changes. Windows is a registered trademark of Microsoft, Inc. Other brand or product names are trademarks of their respective holders.

Compliance Information

For FCC (USA)

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

For IC (Canada)

CAN ICES-3(B)/NMB-3(B)

For CE (EU)

The device complies with the EMC Directive 2014/30/EU and Low Voltage Directive 2014/35/EU

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Usage Notice

Warning - To prevent the risk of fire or shock hazards, and do not expose the product to moisture.

Warning - Please do not open or disassemble the product as this may cause electric shock.

Warning - Power cord shall be connected to a socket-outlet with earthing connection.

Warning - The cable cover cannot be removed under normal use conditions.

Warning - **Stability Hazard.** The touch monitor may fall, causing serious personal injury or death. To prevent injury, this touch monitor must be securely attached to the wall in accordance with the installation instructions.

Precautions

Please follow all warnings, precautions and maintenance as recommended in this user's manual to maximize the life of your unit.

Do :

- Turn off the product before cleaning.
- Use a soft cloth moistened with mild detergent to clean the product housing.
- Use only the qualified power adapter that comes with your device.
- Disconnect the power plug from AC outlet if the product is not going to be used for an extended period of time.

Don't :

- Do not use abrasive cleaners, waxes or solvents for your cleaning.
- Do not operate the product under the following conditions:
 - Extremely hot, cold or humid environment.
 - Areas susceptible to excessive dust and dirt.
 - Near any appliance generating a strong magnetic field.

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Product Introduction

1.1 Overview

The DT-215P-M1 series is a 21.5" touchscreen monitor that builds to withstand commercial grade open frame monitor, with stylish thin cable management bracket and versatile design DT-215P-M1 is an exceptional choice for applications for all business sectors and well suited for point-of-sales, point-of-information, point-of-service and interactive signage.


1.2 Feature

- Cable management bracket design for easy organize cable.


1.3 Specifications

LCD Touch Panel	
Size	21.5" TFT LCD
Brightness	350 cd/m ² (Non-touch screen) 297 cd/m ² (P-cap touch with AG Coating)
Number of Pixels	1920 (H) × 1080 (V)
Touch Type	P-CAP/ 10 points
Environment	
Certificate	E, CB, FCC, UL, RoHS IEC60601-1-2 4th edition compliant
Compliance	Front Panel IP54
Operating Temperature	0°C ~ 40°C
Storage Temperature	-20°C ~ 60°C
Operating Humidity	20% ~ 80% RH, non-condensing
Mounting	VESA 100 mm x 100 mm
Dimension (W x H x D)	517.4 mm x 351.7 mm x 219.0 mm
Net Weight	7.8
Gross Weight	10.8

1.4 Block Diagram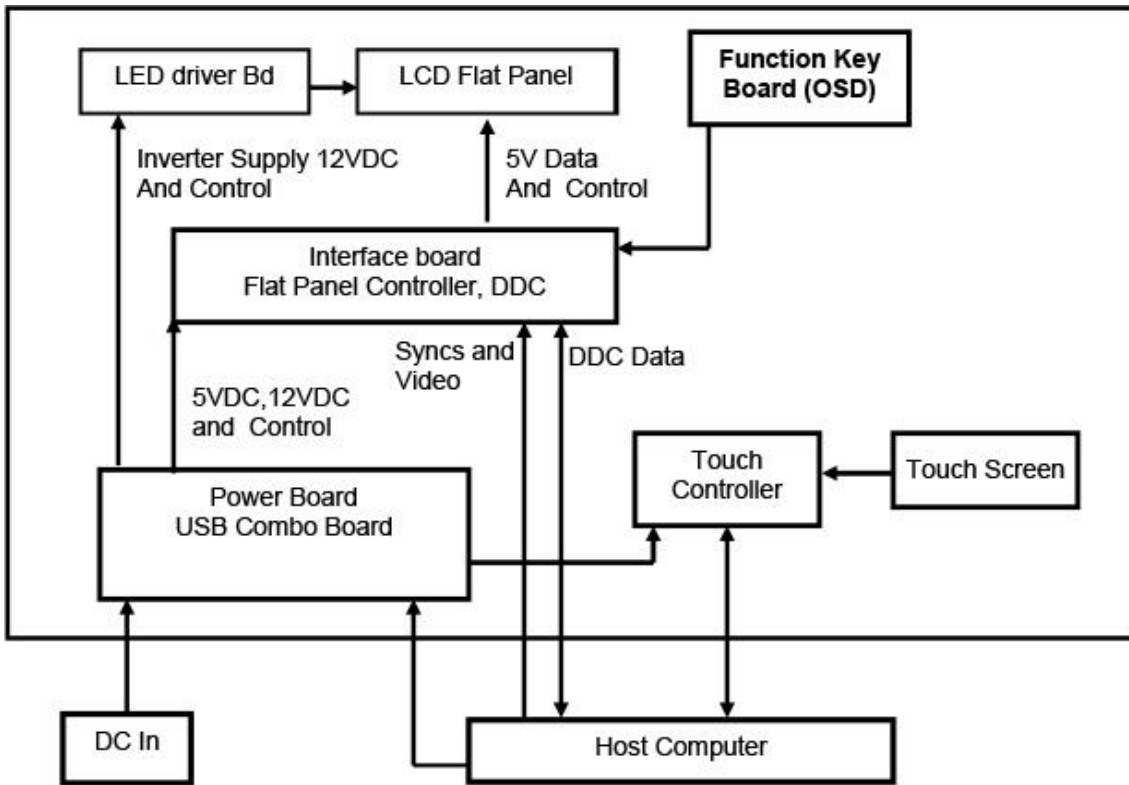


1.5 Interface Connectors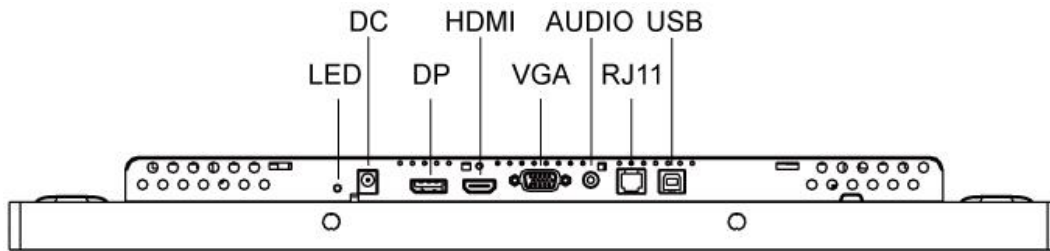

1.5.1 Power Connector

The DC power via external 12V Adapter provide it

1.5.2 Video Signal Connector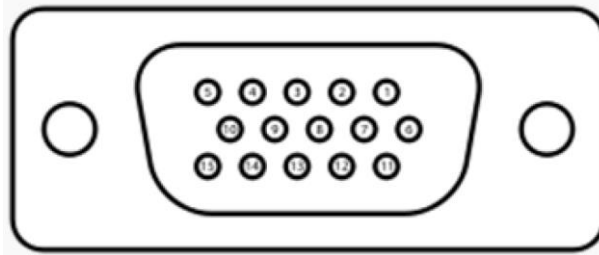


VGA

The video signal input via D-type 15-pin female connector.


Connector Pin Assignment:

Pin	Signal
1	RED
2	GREEN
3	BLUE
4	NC
5	GND
6	RED_RTN
7	GREEN_RTN
8	BLUE_RTN
9	+5V
10	GND
11	NC
12	SDA
13	HSYNC
14	VSYNC
15	SCL

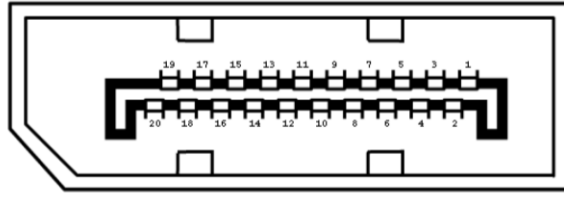Display Port(1.2a)


Pin	Signal	Pin	Signal
1	ML_Lane 0(p) Data0 +	11	Signal ground
2	Signal ground	12	ML_Lane 3(n) Data3 -
3	ML_Lane 0(n) Data0 -	13	Signal ground
4	ML_Lane 1(p) Data1 +	14	Signal ground
5	Signal ground	15	AUX_CH(p) AUX + Signal for Auxiliary Channel
6	ML_Lane 1(n) Data1 -	16	Signal ground
7	ML_Lane 2(p) Data2 +	17	AUX_CH(n) AUX - Signal for Auxiliary Channel
8	Signal ground	18	Hot Plug
9	ML_Lane 2(n) Data 2 -	19	DP_PWR Return
10	ML_Lane 3(p) Data3 +	20	DP_PWR

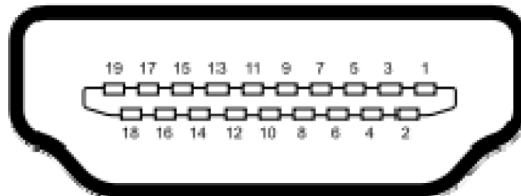HDMI(1.3) (compatible with HDMI1.4)


Pin	Signal	Pin	Signal
1	TMDS Data2+	11	TMDS Clock Shield
2	TMDS Data2 Shield	12	TMDS Clock-
3	TMDS Data2-	13	CEC
4	TMDS Data1+	14	Reserved (N.C. on device)
5	TMDS Data1 Shield	15	SCL
6	TMDS Data1-	16	SDA
7	TMDS Data0+	17	DDC/CEC Ground
8	TMDS Data0 Shield	18	+5V Power
9	TMDS Data0-	19	Hot Plug Detect
10	TMDS Clock+		

1.5.3 Signal Connector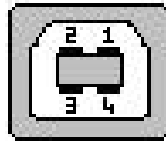


USB Connector

Pin	Signal	Pin	Signal
1	VCC	3	D+
2	D-	4	GND

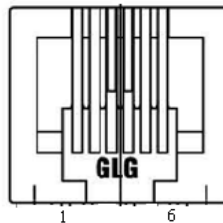RJ11 Connector (for Remote Key) (option)




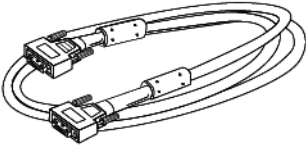
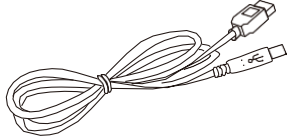
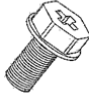
Pin	Signal	Pin	Signal
1	MENU	4	SELECT
2	UP	5	POWER
3	DOWN	6	GND

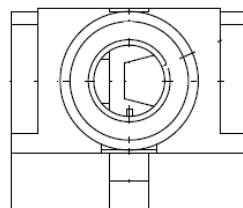Line-in Connector

Pin	Signal	Pin	Signal
1	GND	4	LEFT
2	LEFT	5	RIGHT
3	RIGHT		

1.6 Package Overview

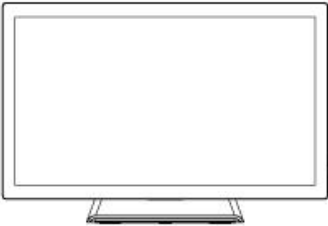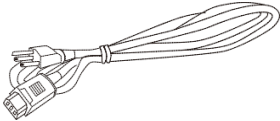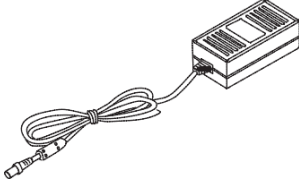		
LCD Display	Power Cord	DC Power Supply
		
HDMI Cable	VGA cable	USB cable (A to B)
		
M3 screw		

Warning!

This product is intended to be supplied by a Listed Power Adapter or DC power source, rated 12Vdc, 4A minimum, Tma = 40 degree C minimum, and the altitude of operation = 3048m minimum. If it needs further assistance with purchasing the power source, please contact to MicroTouch for further information.

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
Product Installation

2.1 About VESA Mount

The DT-215P-M1 series conform to the “VESA Flat Display Mounting Interface Standard” which defines a physical mounting interface for touch monitor, and corresponding with the standards of touch monitor mounting devices. The VESA mount is located on the back of this unit.

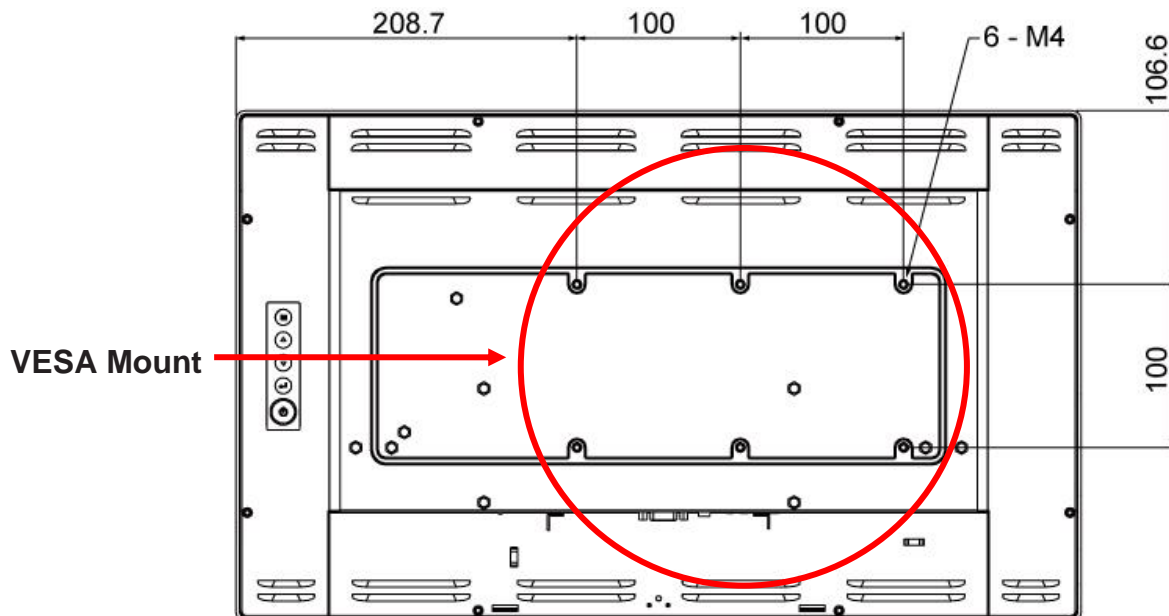Warning!

Please select the MicroTouch original screws!

The distance between the back cover surface and the bottom of the screw hole is 8 mm. Please use four M4 screws diameter with 8-10mm proper length to mount your monitor.

Note: The mounting stand must be able to support at least 7.67 lbs (3.48 Kg).

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2.2 About Wall Mount


The DT-215P-M1 series support wall mount, and this monitor can be mounted to the wooden or concrete wall with 8xM4 screws diameter with 6mm length and two bracket set on the opposites.

2.2.1 Landscape

Depending on your mounting scheme, alternately, you may use the side brackets to mount your displays onto your wall or device.

Find a suitable location for attaching side brackets.

- A. The left and right sides
- B. The Top and bottom sides

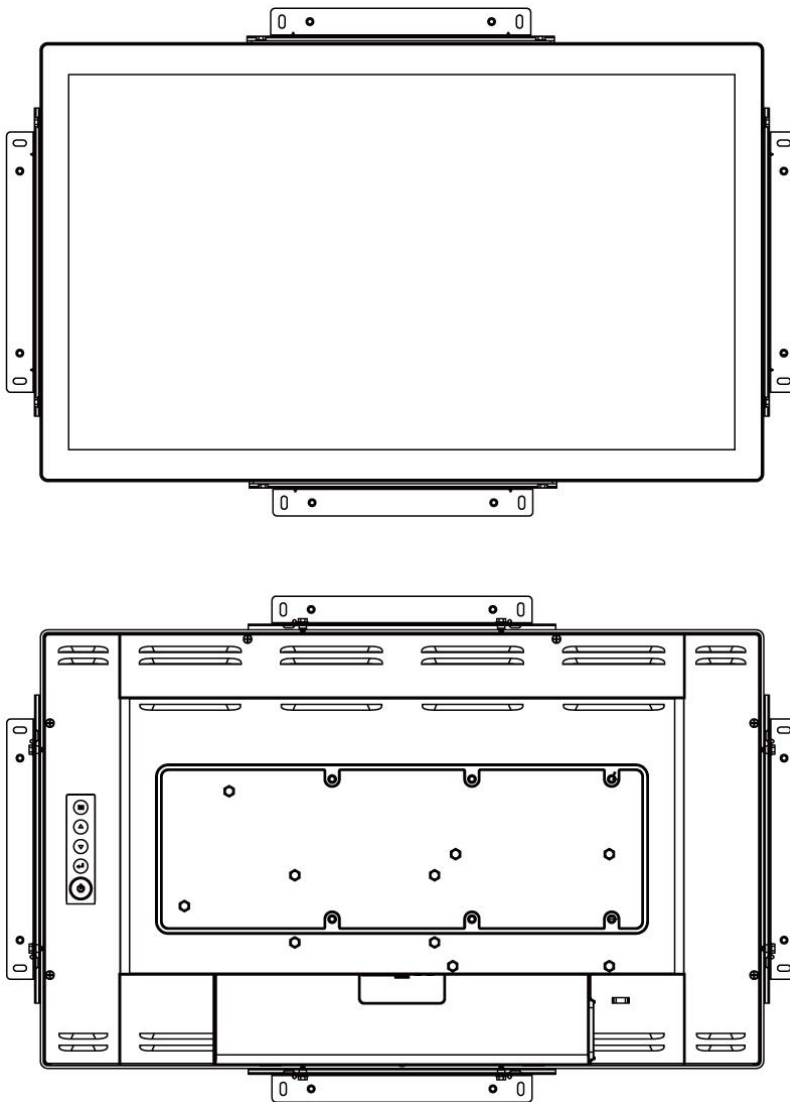Warning!

This apparatus is intended to be supported by a Listed Wall Mount Bracket” or the equivalent.

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


- A. The left and right sides
- B. The Top and bottom sides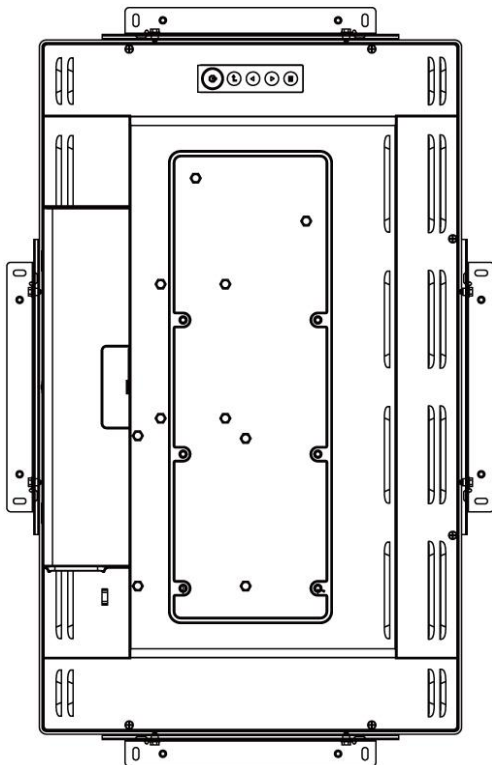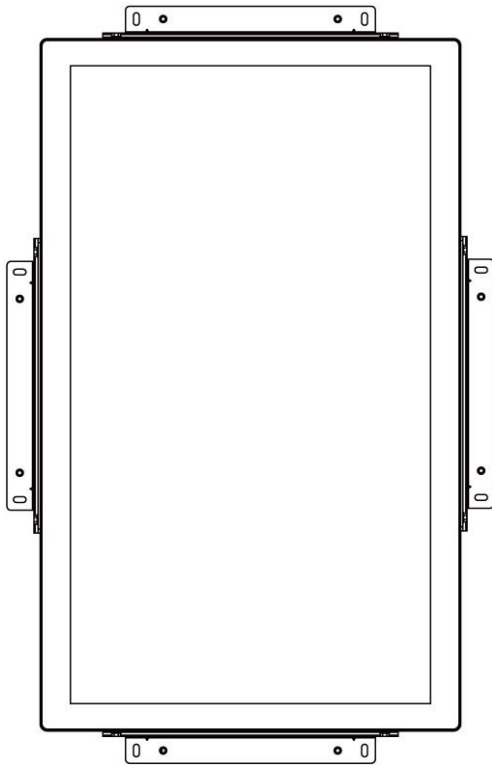


2.2.3 Face up

Step1: Fixed the 4 x L-bracket on the 4-side of DT-215P-M1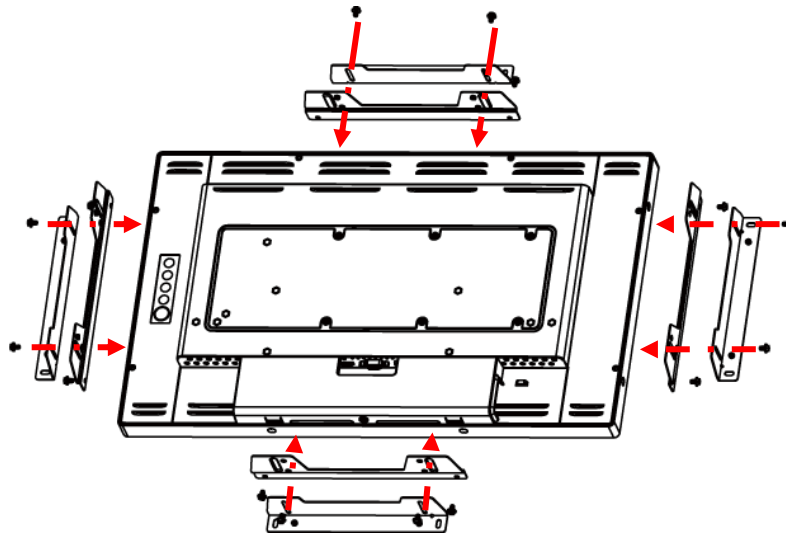


Back side

Step2: Assembly DT-215P-M1 into the wall hole and insert screws.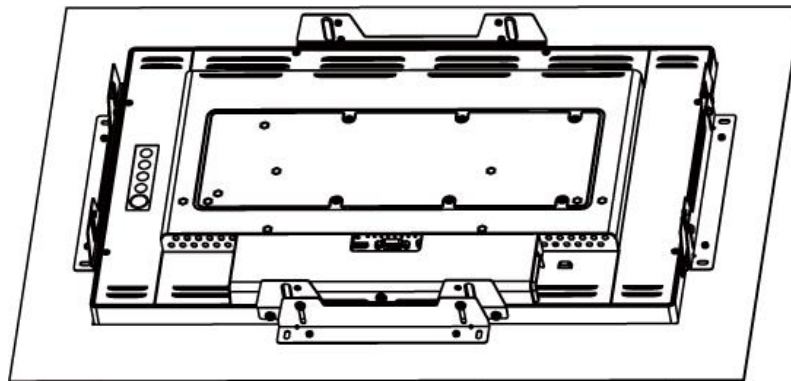


Back side

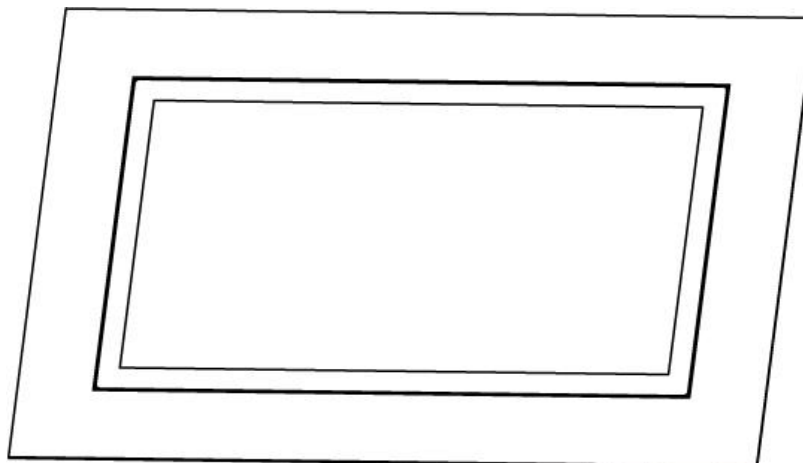Front side

2.3 On-Screen Display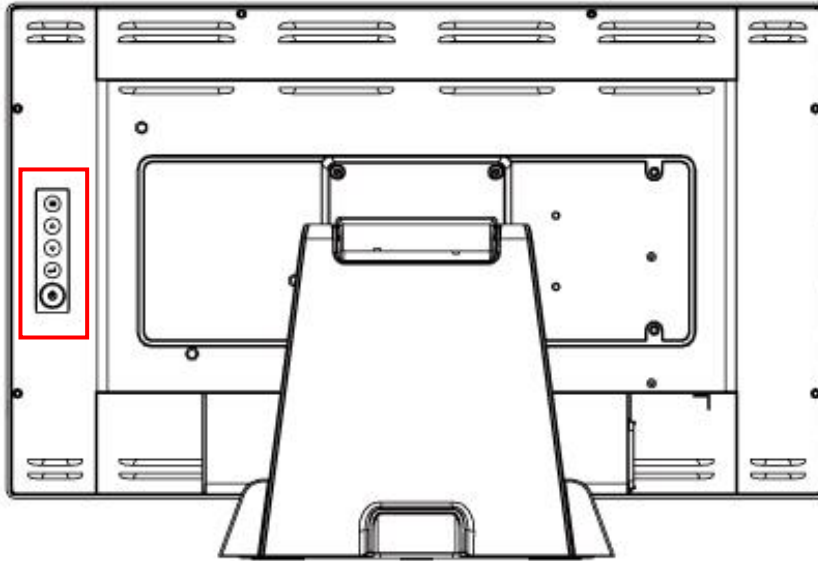

Key \ OSD	Menu off status	Menu on status
☰	Menu appear	Menu disappear/ return to main item
▲	Brightness	Main item select up/ Adjust up
▼	Mute	Main item select down/ Adjust down
↵	Enter/Select sub-item function	
⏻	Power On/Off	

1. Press the “MENU” button to pop up the “on-screen menu” and press “Up” or “Down” button to select among the four functions in the main menu.
2. Choose the adjustment items by pressing the “SELECT ” button.
3. Adjust the value of the adjustment items by pressing the “Up” or “Down” button.
4. With the OSD menu on screen, press “ Menu” button to return main menu or exit OSD.
5. The OSD menu will automatically close, if you have left it idle for a pre-set time.
6. To Lock / Unlock the OSD / Power menu buttons, press “Menu” and “Minus” at the same time to select.

OSD / Power the Lock / Unlock function.

Please note:

- a. When the OSD Lock function is selected, this indicates that all the buttons except "power" button are now disabled.
 - b. When the Power Lock function is selected, this indicates that the power key is disabled; user can not to turn off the monitor by "Power" key.
7. To disable the touch function, press "Menu" and "Enter" Key, at the same time.
8. To enable the touch function, press "Menu" and "Enter" Key, at the same time for five seconds.
9. Press the power key button, there will be an OSD window pop up to ask if it should be going off.
If press the power key in five seconds, the monitor power will be off. If don't press the power key in five seconds, the OSD window will be gone and return to the normal status instead.
10. Direct input select function.
 - a. Normal: Press and keep pressing "Select" key for 3 seconds, will change to pop up "source select" OSD.
 - b. Sleep status: Press and keep pressing "Select" key for 1 seconds, will change to pop up "source select" OSD.
11. Factory mode: Press the "MENU" button to pop up the "on-screen menu" and select "Option" icon in "Factory" item, key in password "3","3","3","3" into factory mode.
12. Burn-in mode: In Factory mode to select "Burn In" item into Burin-in mode (unplug signal cable or do not enter the signal source).
13. Work time: In factory mode, displays monitor the work time counting in hours.

2.3.1 OSD Function Description

Item	Content	Default
Contrast	The monitor luminance level control.	50
Brightness	The monitor backlight level control.	100
H-Positon	Moving screen image horizontal position to left or right.	NA
V-Positon	Moving screen image vertical position to up or down.	NA
Phase	The screen image horizontal dot clock adjustment.	NA
Clock	The screen image pixel phase adjustment.	NA
Auto Adjust	Fine-tune the image to full screen automatically.	NA
Color Temp	Color temperature selection. (9300K, 6500K, 5500K, 7500K, User)	<ul style="list-style-type: none"> ▪ USER ▪ Medical models: R:100,G:93,B:100 ▪ Other models: R:100,G:100,B:100
Medical Models: Sharpness	Adjust the screen sharpness.	50
Medical models: Gamma	Gamma / DICOM function mode. (Native, 1.8, 2.2, DICOM)	Native
OSD Timeout	OSD auto-disappear time selection.	15
Medical models: OSD Positon	Moving OSD menu position.	User
OSD H-Positon	Moving OSD menu horizontal position to left or right.	50
OSD V-Positon	Moving OSD menu vertical position to up or down.	50
Reset	Factory default value restored.	NA
OSD Language	OSD menu language selection. (English, French, Deutsch, Italian, Spanish, Japanese, Traditional Chinese and Simplified Chinese)	English
Source Select	Select Input function. (Auto, VGA,HDMI,DP)	Auto
Medical models: Display port	Display current resolution related information.	NA
Volume	Audio volume adjustment.	50
Mute	Audio On/Off control.	Off
Medical models: Lum Preset	The monitor backlight function quick mode. (Set1, Set2, Set3, User)	User
Factory	Factory mode related adjustment items.	NA

2.3.2 Timing Table Chart

Mode	Resolution	H-Freq. (KHz)	Band Width (MHz)	Polarity	
				H	V
1	VGA 720x400 70Hz	31.47	28.322	-	+
2	VGA 640x480 60Hz	31.47	25.175	-	-
3	MAC 640x480 66Hz	35	32.24	-	-
4	VESA 640x480 72Hz	37.86	31.5	-	-
5	VESA 640X480 75Hz	37.5	31.5	-	-
6	VESA 800x600 56Hz	35.16	36	+	+
7	VESA 800x600 60Hz	37.88	40	+	+
8	VESA 800x600 75Hz	46.88	49.5	+	+
9	VESA 800x600 72Hz	48.08	50	+	+
10	MAC 832x624 75Hz	49.72	57.283	-	-
11	VESA 1024x768 60Hz	48.36	65	-	-
12	VESA 1024x768 70Hz	56.48	75	-	-
13	VESA 1024x768 75Hz	60.02	78.75	+	+
14	SXGA 1280x1024 60Hz	64	108	+	+
15	SXGA 1280x1024 75Hz	80	135	+	+
16	SXGA 1152x864 75Hz	67.5	108	+	+
17	SXGA 1280x960 60Hz	60	108	+	+
18	WXGA+ 1440x900 60Hz	56	106.5	-	+
19	WXGA+ 1440x900 75Hz	70.6	136.75	-	+
20	WSXGA+ 1680x1050 60Hz	65.2	146	-	+
21	WSXGA+ 1680x1050 75Hz	82.3	187	-	+
22	VESA 1280x 768 60Hz	47.776	79.5	-	+
23	1920X1080 60Hz	67.5	148.5	+	+

2.3.3 EDID Data

DP/HDMI


The monitor assembly shall provide a display communications channel that conforms to VESA DDC2B hardware requirements. This configuration shall contain the 256-byte EDID file as specified by VESA EDID Standard

VGA


The monitor assembly shall provide a display communications channel that conforms to VESA DDC2B hardware requirements. This configuration shall contain the 128-byte EDID file as specified by VESA EDID Standard

2.4 Dimension



2.4.1 Front View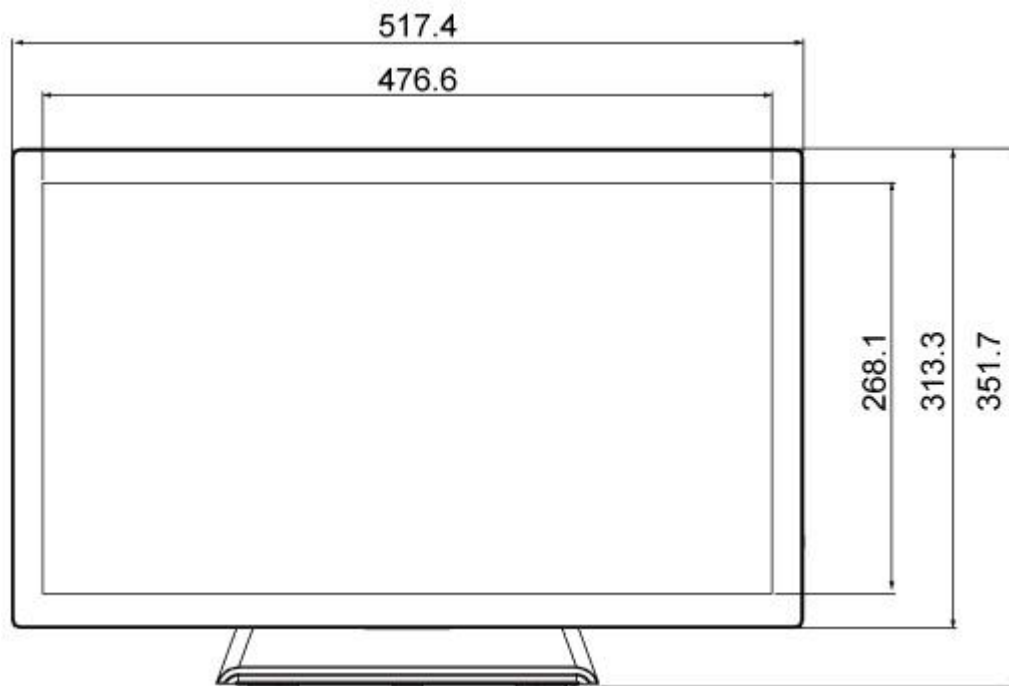

2.4.2 Side View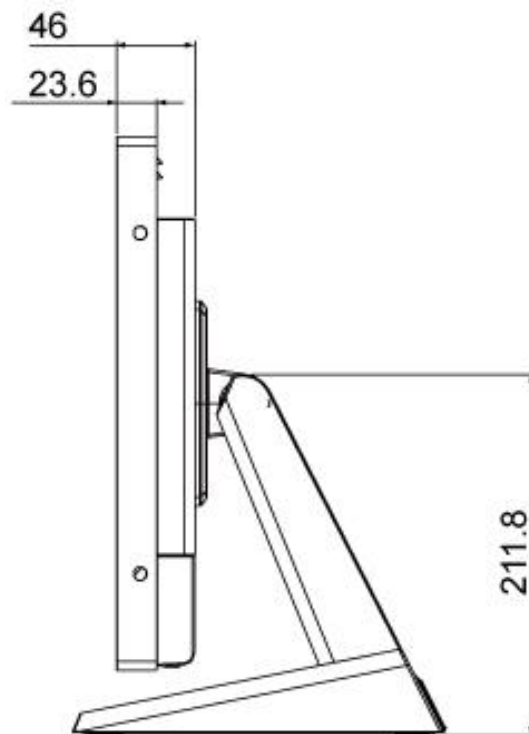

2.4.3 Rear View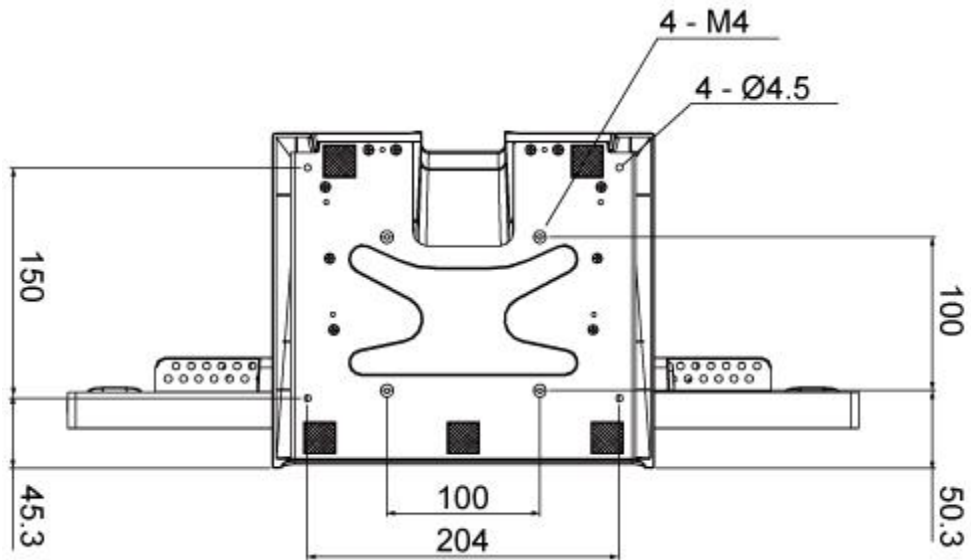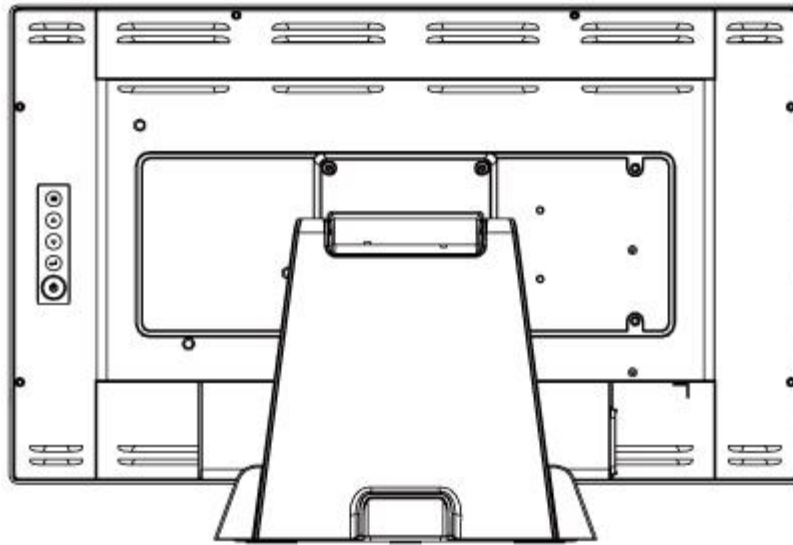

Appendix

Declaration of the Presence Condition of the Restricted Substances Marking

設備名稱：觸控螢幕 Equipment name : Touch LCD Monitor		型號（型式）：DT-215P-M1 Type designation (Type) : DT-215P-M1				
單元 Unit	限用物質及其化學符號 Restricted substances and its chemical symbols					
	鉛 Lead (Pb)	汞 Mercury (Hg)	鎘 Cadmium (Cd)	六價鉻 Hexavalent chromium (Cr ⁺⁶)	多溴聯苯 Polybrominated biphenyls (PBB)	多溴二苯醚 Polybrominated diphenyl ethers (PBDE)
塑膠零件 Plastic Parts	○	○	○	○	○	○
金屬零件 Metal Parts	-	○	○	○	○	○
線纜和電纜組件 Cable component	-	○	○	○	○	○
LCD 面板 LCD Panel	-	○	○	○	○	○
觸控式螢幕面板 Touch Panel	-	○	○	○	○	○
PCBA	-	○	○	○	○	○
軟體(CD 等) Software	○	○	○	○	○	○
<p>Note 1 : “Exceeding 0.1 wt %” and “exceeding 0.01 wt %” indicate that the percentage content of the restricted substance exceeds the reference percentage value of presence condition. °</p> <p>Note 2 : “○” indicates that the percentage content of the restricted substance does not exceed the percentage of reference value of presence.</p> <p>Note 3 : The “-” indicates that the restricted substance corresponds to the exemption.</p>						